

**Assembly Instructions Pelvi-Loc 2DS / 3DS / 2DAS / 3DAS**

**Picture 1:** Lead the webbing around the tube

**Picture 2:** Drill through the middle of the webbing with a graver or soldering iron

**Picture 3:** M6 screw with washer through the belt strap Screw and washer and nut tighten

**Picture 4:** Tighten the washer with the nut on the back of the belt

**Picture 1**




**Picture 2**



**Picture 3**



**Picture 4**

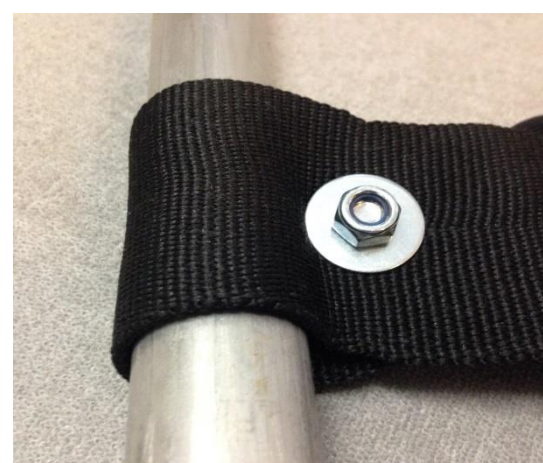

The webbing can also be punched several times, keep a distance of at least 2 cm from hole to hole.